

FACT SHEET FOR HEALTHCARE PROVIDERS

Laboratory Corporation of America (Labcorp)
Labcorp SARS-CoV-2 & Influenza A/B Assay

September 30, 2021

Coronavirus
Disease 2019
(COVID-19)

This Fact Sheet informs you of the significant known and potential risks and benefits of the emergency use of the Labcorp SARS-CoV-2 & Influenza A/B Assay.

The Labcorp SARS-CoV-2 & Influenza A/B Assay is authorized for use with anterior nasal swab specimens collected at home by individuals suspected of respiratory viral infection consistent with COVID-19 by a healthcare provider.

All patients whose specimens are tested with this assay will receive the Fact Sheet for Patients: Labcorp - Labcorp SARS-CoV-2 & Influenza A/B Assay.

What are the signs and symptoms of COVID-19?

Many patients with COVID-19 have developed fever and/or symptoms of acute respiratory illness (e.g., cough, dyspnea), although some individuals experience only mild symptoms or no symptoms at all. The current information available to characterize the spectrum of clinical illness associated with COVID-19 suggests that, when present, symptoms include cough, shortness of breath or dyspnea, fever, chills, myalgias, headache, sore throat, new loss of taste or smell, nausea or vomiting or diarrhea. Signs and symptoms may appear any time from 2 to 14 days after exposure to the virus, and the median time to symptom onset is approximately 5 days. For further information on the symptoms of COVID-19 please see the link provided in "*Where can I go for updates and more information?*" section.

Public health officials have identified cases of COVID-19 infection throughout the world, including the United States. Please check the CDC COVID-19 webpage (see link provided in "*Where can I go for updates and more information?*" section at the end of this document) or your local jurisdictions website for the most up to date information.

What are the signs and symptoms of influenza?

The signs and symptoms of influenza usually develop suddenly and are similar to those of COVID-19. Common signs and symptoms of influenza are fever,

This test is to be performed only using anterior nasal swab specimens collected at home by individuals suspected of respiratory viral infection consistent with COVID-19 by a healthcare provider.

cough, sore throat, runny/stuffy nose, body aches, headache, and fatigue.

What do I need to know about COVID-19 testing?

Current information on COVID-19 for healthcare providers is available at CDC's webpage, *Information for Healthcare Professionals* (see links provided in "*Where can I go for updates and more information?*" section). The Labcorp SARS-CoV-2 & Influenza A/B Assay:

- can be used to test anterior nasal swab specimens collected at home,
- should be ordered for the detection of COVID-19 in individuals suspected of respiratory viral infection consistent with COVID-19 by a healthcare provider (HCP),
- is for use with anterior nasal swab specimens that are collected by individuals age 18 years and older (self-collected), 14 years and older (self-collected under adult supervision), or 2 years and older (collected with adult assistance) using either the Labcorp COVID-19 + Flu Test Home Collection Kit or the Pixel by Labcorp COVID-19 + Flu Test Home Collection Kit when ordered directly by an HCP,
- is only authorized for use at Center for Esoteric Testing, Burlington, NC, or other laboratories designated by Labcorp that are also certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a, and meet the requirements to perform high complexity tests.

Specimens should be collected with appropriate infection control precautions. Current guidance is available at the CDC's website (see links provided in "*Where can I go for updates and more information?*" section).

When collecting and handling specimens from individuals suspected of being infected with COVID-19, appropriate personal protective equipment should be

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

FACT SHEET FOR HEALTHCARE PROVIDERS

Laboratory Corporation of America (Labcorp)
Labcorp SARS-CoV-2 & Influenza A/B Assay

September 30, 2021

Coronavirus
Disease 2019
(COVID-19)

used as outlined in the CDC *Interim Laboratory Biosafety Guidelines for Handling and Processing Specimens Associated with Coronavirus Disease 2019 (COVID-19)*. For additional information, refer to CDC *Interim Guidelines for Collecting, Handling, and Testing Clinical Specimens from Persons Under Investigation (PUIs) for Coronavirus Disease 2019 (COVID-19)* (see links provided in “Where can I go for updates and more information?” section).

What does it mean if the specimen tests positive for SARS-CoV-2, the virus that causes COVID-19?

A positive test result for COVID-19 indicates that RNA from SARS-CoV-2 was detected, and therefore the patient is infected with the virus and presumed to be contagious. Laboratory test results should always be considered in the context of clinical observations and epidemiological data (such as local prevalence rates and current outbreak/epicenter locations) in making a final diagnosis and patient management decisions. Patient management should be made by a healthcare provider and follow current CDC guidelines.

The Labcorp SARS-CoV-2 & Influenza A/B Assay has been designed to minimize the likelihood of false positive test results. However, it is still possible that this test can give a false positive result, even when used in locations where the prevalence is below 5%. In the event of a false positive result, risks to patients could include the following: a recommendation for isolation of the patient, monitoring of household or other close contacts for symptoms, patient isolation that might limit contact with family or friends and may increase contact with other potentially COVID-19 patients, limits in the ability to work, delayed diagnosis and treatment for the true infection causing the symptoms, unnecessary prescription of a treatment or therapy, or other unintended adverse effects.

All laboratories using this test must follow the standard testing and reporting guidelines according to their appropriate public health authorities.

What does it mean if the specimen tests negative for SARS-CoV-2, the virus that causes COVID-19?

A negative test result for SARS-CoV-2 means that RNA from this virus was not present in the specimen or the

RNA concentration was below the limit of detection. However, a negative result does not rule out COVID-19 and should not be used as the sole basis for treatment or patient management decisions. It is possible to test a person too early or too late during COVID-19 infection to make an accurate diagnosis via the Labcorp SARS-CoV-2 & Influenza A/B Assay.

Specimens that are self-collected will not be tested with an internal control to confirm that the specimen was properly collected. As such, unobserved self-collected specimens from SARS-CoV-2 positive individuals may yield negative results if the specimen was not collected properly.

When diagnostic testing results are negative, the possibility of a false-negative result should be considered in the context of a patient’s recent exposures and the presence of clinical signs and symptoms consistent with COVID-19. The possibility of a false-negative result should especially be considered if the patient’s recent exposures or clinical presentation indicate that COVID-19 is likely, and diagnostic test results for other causes of illness (e.g., other respiratory illnesses) are negative. If COVID-19 is still suspected based on exposure history and clinical findings, retesting should be considered by healthcare providers in consultation with public health authorities. Additional testing may be helpful to ensure testing was not conducted too early. Risks to an individual from a false-negative test result include: delayed or lack of supportive treatment; lack of monitoring of infected patients and their household or other close contacts for symptoms, resulting in increased risk of spread of COVID-19 within the community; or other unintended adverse events.

The performance of this test was established based on the evaluation of a limited number of clinical specimens. The clinical performance has not been established in all circulating variants but is anticipated to be reflective of the prevalent variants in circulation at the time and location of the clinical evaluation. Performance at the time of testing may vary depending on the variants circulating, including newly emerging strains of SARS-CoV-2 and their prevalence, which change over time.

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

FACT SHEET FOR HEALTHCARE PROVIDERS

Laboratory Corporation of America (Labcorp)
Labcorp SARS-CoV-2 & Influenza A/B Assay

September 30, 2021

Coronavirus
Disease 2019
(COVID-19)

What does it mean if the specimen tests positive for influenza A and/or B viruses?

A positive test result for influenza A virus or influenza B virus indicates that RNA from one or both of these viruses was detected, the patient is infected with the virus(es) and is presumed to be contagious. Laboratory test results should always be considered in the context of clinical findings and observations and epidemiological data in making a final diagnosis. Patient management decisions should be made by a healthcare provider and follow current CDC guidelines. Results (positive and negative) for influenza should be interpreted with caution. If a result is inconsistent with clinical presentation and/or other clinical and epidemiological information, FDA-cleared influenza NAATs are available for confirmation if clinically indicated.

The Labcorp SARS-CoV-2 & Influenza A/B Assay has been designed to minimize the likelihood of false-positive test results. However, in the event of a false-positive result, risks to individuals could include the following: a recommendation for isolation of the patient, monitoring of household or other close contacts for symptoms, patient isolation that might limit contact with family or friends, limits in the ability to work, delayed diagnosis and treatment for the true infection causing the symptoms, unnecessary prescription of an antiviral medication or other therapy, or other unintended adverse effects.

What does it mean if the specimen tests negative for influenza viruses?

A negative test result for influenza viruses means that influenza A and/or B RNA was not present in the specimen or the RNA concentration was below the limit of detection. However, a negative result does not rule out influenza virus infection and should not be used as the sole basis for treatment or patient management decisions. When diagnostic testing results are negative, the possibility of a false-negative result should be considered in the context of a patient's recent exposures and the presence of clinical signs and symptoms consistent with influenza. The possibility of a false-negative result should especially be considered if the patient's recent exposures or clinical presentation indicate that influenza is likely, and diagnostic test results for other causes of illness (e.g., other respiratory

illness) are negative. If influenza is still suspected based on exposure history and clinical findings, re-testing should be considered by healthcare providers in consultation with public health authorities.

Laboratory test results should always be considered in the context of clinical findings and observations and/or epidemiological data in making a final diagnosis. Patient management decisions should be made by a healthcare provider and follow current CDC guidelines. Results (positive and negative) for influenza should be interpreted with caution. If an influenza result is inconsistent with clinical presentation and/or other clinical and epidemiological information, FDA-cleared Influenza NAATs are available for confirmation if clinically indicated.

Risks to individuals from a false-negative Labcorp SARS-CoV-2 & Influenza A/B Assay result for influenza A or influenza B include: delayed or lack of supportive treatment; lack of monitoring of infected patients and their household or other close contacts for symptoms, resulting in increased risk of spread of influenza within the community; or other unintended adverse events. Risks to an asymptomatic patient from a false-negative Labcorp SARS-CoV-2 & Influenza A/B Assay result for influenza A or B include: lack of monitoring of infected patients and their household or other close contacts for symptoms, resulting in increased risk of spread of influenza within the community; or other unintended adverse events.

What does it mean if the specimen tests positive for SARS-CoV-2 and one or both influenza (A and/or B) viruses? Is co-infection possible?

Yes, it is possible for an individual to be infected with influenza A virus, influenza B virus, and/or SARS-CoV-2 simultaneously. A positive test result for the viruses that cause COVID-19 and influenza A and/or B indicates that RNA from these viruses was detected, the patient may be co-infected, and is presumed to be contagious. Laboratory test results should always be considered in the context of clinical findings and observations and epidemiological data in making a final diagnosis. Patient management decisions should be made with a healthcare provider and follow current CDC guidelines.

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

FACT SHEET FOR HEALTHCARE PROVIDERS

Laboratory Corporation of America (Labcorp)
Labcorp SARS-CoV-2 & Influenza A/B Assay

September 30, 2021

Coronavirus
Disease 2019
(COVID-19)

What is an EUA?

The United States FDA has made this test available under an emergency access mechanism called an Emergency Use Authorization (EUA). The EUA is supported by the Secretary of Health and Human Service's (HHS's) declaration that circumstances exist to justify the emergency use of in vitro diagnostics (IVDs) for the detection and/or diagnosis of the virus that causes COVID-19.

An IVD made available under an EUA has not undergone the same type of review as an FDA-approved or cleared IVD. FDA may issue an EUA when certain criteria are met, which includes that there are no adequate, approved, available alternatives, and based on the totality of scientific evidence available, it is reasonable to believe that this IVD may be effective in diagnosing COVID-19.

The EUA for this test is in effect for the duration of the COVID-19 declaration justifying emergency use of IVDs, unless terminated or revoked (after which the test may no longer be used).

What are the approved available alternatives?

FDA has approved/cleared certain influenza tests. Any tests that have received full marketing status (e.g., cleared, approved), as opposed to an EUA, by FDA can be found by searching the medical device databases here: <https://www.fda.gov/medical-devices/deviceadvice-comprehensive-regulatory-assistance/medicaldevice-databases>. A cleared or approved test should be used instead of a test made available under an EUA, when appropriate and available. FDA has issued EUAs for other tests that can be found at: <https://www.fda.gov/emergency-preparedness-and-response/mcm-legal-regulatory-and-policy-framework/emergency-use-authorization>.

Where can I go for updates and more information?

CDC webpages:

General: <https://www.cdc.gov/coronavirus/2019-ncov/index.html>

Symptoms:

<https://www.cdc.gov/coronavirus/2019-ncov/symptoms-testing/symptoms.html>

Healthcare Professionals:

<https://www.cdc.gov/coronavirus/2019-nCoV/hcp/index.html>

Information for Laboratories:

<https://www.cdc.gov/coronavirus/2019-nCoV/lab/index.html>

Laboratory Biosafety: <https://www.cdc.gov/coronavirus/2019-nCoV/lab-biosafety-guidelines.html>

Isolation Precautions in Healthcare Settings:

<https://www.cdc.gov/infectioncontrol/guidelines/isolation/index.html>

Specimen Collection: <https://www.cdc.gov/coronavirus/2019-nCoV/guidelines-clinical-specimens.html>

Infection Control: <https://www.cdc.gov/coronavirus/2019-ncov/php/infection-control.html>

FDA webpages:

General: www.fda.gov/novelcoronavirus

EUAs:(includes links to patient fact sheet and manufacturer's instructions) <https://www.fda.gov/medical-devices/coronavirus-disease-2019-covid-19-emergency-use-authorizations-medical-devices/in-vitro-diagnostics-euas>

Laboratory Corporation of America:

Laboratory Corporation of America Holdings
531 S Spring St
Burlington, NC 27215

Customer Support:

LabCorp COVID-19 + Flu Test Home Collection Kit:

+1 (800) 222-7566

Covid19request@labcorp.com

Customer Support

Pixel by LabCorp COVID-19 + Flu Test Home Collection Kit:

+1 (800) 833-3935

support@pixel.Labcorp.com

Technical Support

Covid19@pwnhealth.com

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**