

[Start transcript

Visual: Image of at-home testing kit appears in front of a solid purple background and beside the title of the video: "Simple Nasal Swab"

Woman wearing a navy-blue blouse and necklace standing in kitchen of a house grabs both a collection vial and single-use, plastic-sealed nasal swab from her kit.

The woman unscrews the top of the collection vial and then places both the collection vial top and vial itself on the table in front of them. The woman then opens the single-use, plastic-sealed nasal swab by holding the clear plastic flap with one hand and pulling the white plastic flap with the other hand in a downward motion.

The woman removes the nasal swab and shows it to the camera then inserts the tip of the nasal swab into one nostril. The woman then rotates the swab in a circular motion around the entire inside edge of the nostril three (3) times and repeats again in the other nostril.

The woman removes the swab from her nostril and immediately places it in the collection vial with the tip side at the bottom of the vial. The woman then screws the top of the vial back in place sealing the sample and then shows the vial to the camera.

Visual: Pixel by Labcorp logo

Audio: Throughout the video upbeat background music without lyrics plays.

End transcript]